

A U D E A R A



A-01 HEADPHONE + BT-01 TV TRANSMITTER

How To Guide

A01 HEADPHONE AND BT01 FEATURES

Hands free Calling

The in-built microphone and clear voice capture technology lets you hear and communicate clearly over the phone, or via video chat.

Bluetooth Compatible Devices

The Audeara headphones will connect to all Bluetooth compatible devices, such as laptops, iPads, tablets and mobile phones.

Connect the BT01 transceiver to your TV

Watch your favourite TV shows or movies with clear, tailored sound through the BT01 TV transceiver.

Personalised Sound

The headphones allow you to calibrate your hearing results directly into the headphones to provide tailored sound. This is done by performing a hearing test on your smartphone using the Audeara App.

Extended battery life

Enjoy up to 35 hours of continuous Bluetooth streaming with Active Noise Cancellation activated.

Active Noise Cancellation

Reduce the impact of unwanted background noise by turning on the Active Noise Cancellation (ANC) feature, available on the right ear cup on your headphones.

Compatible with Hearing Aids and Cochlear Implants

Wear your A01 Wireless Headphones over the top of your Hearing Aids or Cochlear Implants to enhance your listening experience.



Volume control and multifunction buttons.

Use the three buttons on the left ear cup to control the volume (the top button will increase volume, button button will reduce volume and the middle button is used to pair the headphones, answer phone calls or pause music/videos).

PAIRING THE A-01 HEADSET TO YOUR PHONE

1. Turn on the headphones using the power switch on the left ear cup.
2. Hold the middle button on the left ear cup of the headphones for three seconds or until the light flashes blue and red.
3. Open your phone and navigate to settings> Bluetooth and make sure it is active.
4. Your phone should display "Audeara A-01" to be selected in your Bluetooth settings.



5. Select "Audeara A-01" to complete pairing.



INSTALLING THE AUDEARA APP

The Audeara headphones are designed to be tailored based on your individual hearing profile. To personalise your headphones, you will need to download and install the Audeara app.


1. **Navigate to the App Store (for iOS iPhones) or to the 'Play Store' (for Android).**
2. **Enter 'Audeara' in the search bar.**
3. **Select 'Install'**
4. **Once it has been installed, select 'Open' to access the App.**

Create your Account:

You can choose to sign in through your Facebook or Google account.

Create an account through email:

1. **Select 'Create Account'**
2. **Complete the fields and enter a password.**
3. **Select 'Create Account'**
4. **Log in to your email to verify your details.**
5. **Go back to the Audeara App.**
6. **Select 'Email Sign in'.**
7. **Enter your email and password.**

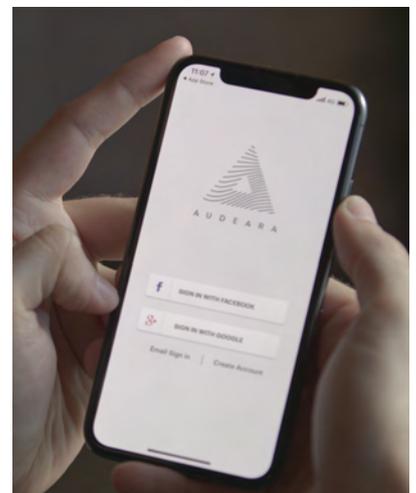

PERFORMING YOUR HEARING TEST

We recommend a quiet space to complete your hearing test.

1. Ensure your A01 headset is turned on and paired to your smartphone.
2. Select 'Start Test'
3. Select 'Continue'
4. Read through the information provided and select 'Continue' to move on.

5. Select your test.

Note: There are three levels of testing. The standard test takes only a few minutes to complete with 8 frequencies per ear. If you prefer a more detailed test, you can select the 'High Detail' or the 'Ultimate Precision'. **If you have a known hearing loss or wear a hearing aid, we recommend the 'Standard' or 'High detail test'. You can wear the headphones over the top of your hearing aids while you perform your test.**

6. Select 'Continue' to navigate through the tutorial.


7. Use the 'Can Hear' and 'Can't Hear' buttons to increase or reduce the volume of the beeps until the beep becomes barely audible, that is the absolute softest sound you can hear.

Note: If you are unable to hear the beep after pressing 'Can't Hear' until it reaches the top of the limitation within the app, press 'Barely Audible' and move to the next frequency. Audeara's technology will recognise this and adjust accordingly.

8. When you can only just hear the beep, select 'Barely Audible' to move onto the next frequency.

9. Select 'Finish Now' to complete your test.

10. Review your results by selecting 'View Hearing Health Profile'.

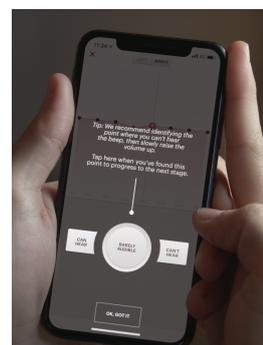


PERSONALISING YOUR HEARING RESULTS

The Audeara app provides the option for the user to select and update the level of personalisation. This provides additional control of your listening experience to select what level of your hearing test results are to be applied to the headphones.

You have the ability to revisit and open up your hearing profile to change the level of personalisation at any time.

1. **Complete your hearing test and view your results (shown above).**
2. **Select 'Experience Audeara'.**
3. **Your hearing results are now being applied to your headphones. This may take a minute to complete.**
4. **For best results, play your favourite song through your connected device into your A01 headphones.**
5. **Navigate back to the Audeara app while music continues to play.**
6. **Select 50% personalisation on the bar, then select 'apply'.**
7. **Continue to move through the levels (selecting 'Apply' each time) to find the level best suited to you.**
8. **Once you are happy with the level, close the Audeara app.**
9. **Your settings will be saved in the headphones and will remain until you apply another hearing profile or adjust the personalisation o**

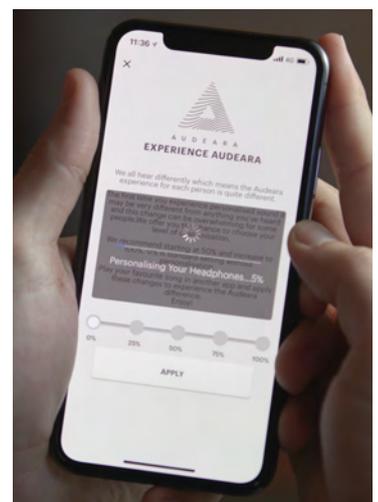

PAIRING THE BT-01 TV TRANSCEIVER

1. Turn on the headphones using the power switch on the left ear cup.
2. Slide the transceiver button on to “TX” on the back to turn on “Transceiver Mode”.



3. Hold the middle button on the left ear cup of the headphones for three seconds or until the light flashes blue and red.
4. Slide the button on the top of the TV transceiver into “pairing mode” until it flashes red and blue.



5. The colour will flash to blue and then become a steady blue light on both devices to show they have been paired.

PLUGGING INTO YOUR TV

1. Look for the “Digital Audio Out” port at the back of your TV. If you do not have this, please skip to step 3.



2. Use the Optical Cable (shown below) to plug one end in the “Digital Audio Out” port and the other in the “Digital Optical Audio In” at the back of the TV Transmitter.



3. If you do not have a “Digital Audio Out” port on your TV. Use the Red and White cable to plug into your TV.
4. Plug the other end of the cable into the “Audio In” port on the transceiver.

Note

The Optical Cord allows the sound to be played out of the Headphones and the TV. If using the red and white cable, the sound will come from the headphones only.



PAIRING TO YOUR PHONE + TV TRANSMITTER.

1. Pair the headphones to the TV transmitter as described.
2. Turn off your headphones and wait 5 seconds.
3. Turn back on and pair with your mobile as described.
4. Your phone should display "Audeara A-01" to be selected in your Bluetooth settings.
5. Turn back off again.

